


air#

Style Air Mask

How to Remove the Air Valve


1

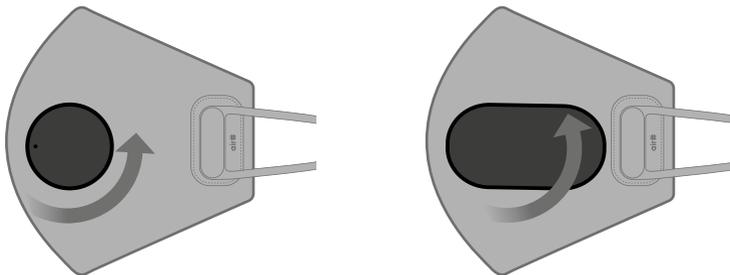

To remove the air valve or the fan unit*, detach by rotating counterclockwise.

*** Fan unit is sold separately.**

2

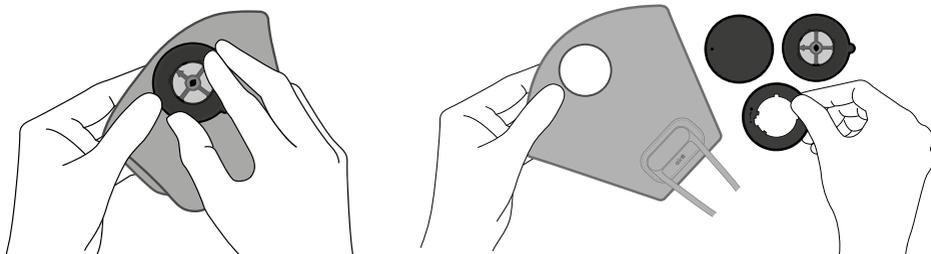


Detach the bracket from the mask to remove all remaining air valve components.

How to Install an Inner Filter


You have the option to either use the outer mask by itself or to install an inner filter.
Remove the air valve → Install the inner filter → Assemble the air valve before use.

1

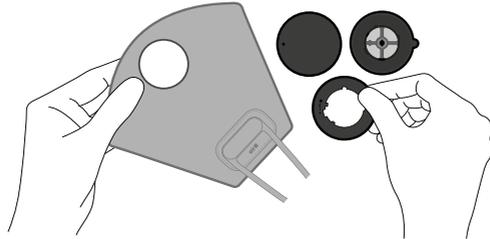

Remove all air valve components.
Refer to: "How to Remove the Air Valve"


2

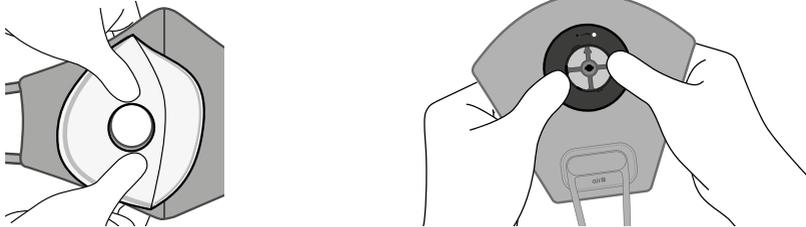


Align both the inner filter and the outer mask by the hole. Assemble the air valve at the hole.
Refer to: "How to Assemble the Air Valve"

- * Prior to installing the internal filter, please ensure to clean hands to prevent any contamination.
- * The inner filter is a consumable part, so regular replacement is required. Our recommendation for use is 1-2 days.
- * Additional inner filters are available for purchase online at our official website.
- * Images used are solely to display instructions and may vary from the actual product.

How to Assemble the Air Valve

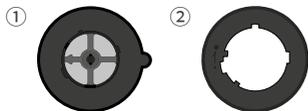


1

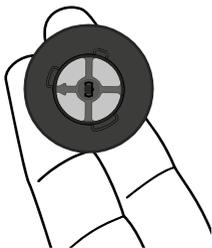


Place Bracket ①
on your index and middle fingers.

3

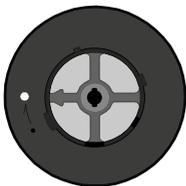

Locate the hole in the Bracket ②.
Align the hole to the red dot.

2


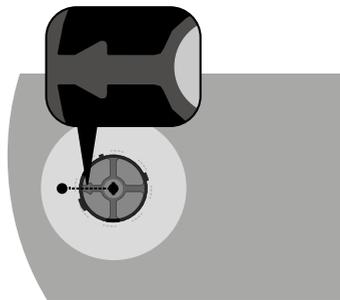


Insert Bracket ①
into the hole from the
inside of the mask. Look for the arrow sign
on the bracket. Ensure the arrow faces
and aligns with the red dot located on the
outside of the mask.

How to Assemble the Air Valve



4

How to Assemble Air Valve Cover

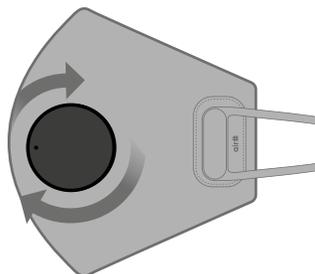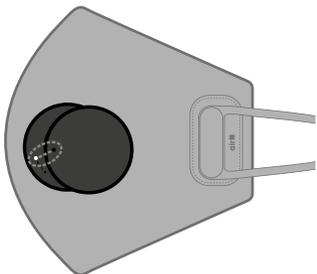


Align the dots on Bracket ② and Air Valve Cover in a linear position.
Spin the Air Valve cover clockwise.

How to Assemble the Fan Unit

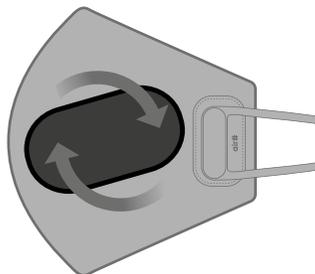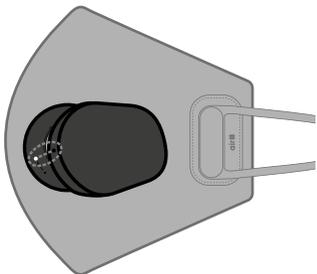

Align the dots on Bracket ② and the Fan Unit in a linear fashion.
Spin the Fan Unit clockwise.

Precautions

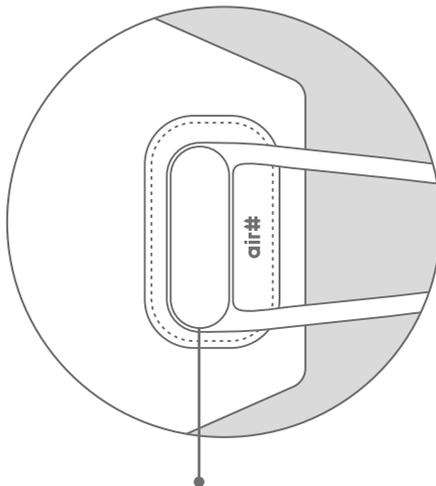

Do not twist the mask while washing or drying.
Ear band may fall off the mask with an excessive force.
Wash the mask gently with your hands and dry the mask with a towel to remove moisture.
Any damage due to consumer negligence will not be compensated.

How to Wash the Mask



HAND
WASH



DO NOT
BLEACH



DO NOT
IRON




DO NOT
DRY CLEAN




DO NOT
WRING




DO NOT
TUMBLE DRY

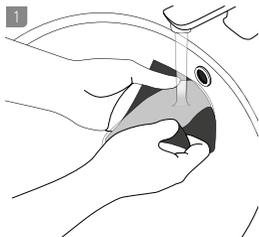


1 Remove the filter and wash the mask in cold water. Please do not soak it in water for a long time. As it may stain the product.

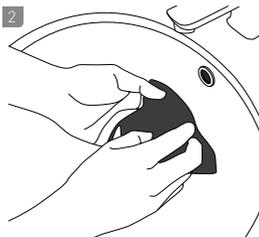


2 Rub the mask gently with neutral detergent. The nose bridge in the mask may be damaged when an excessive force is applied. Please rinse off the detergent thoroughly.

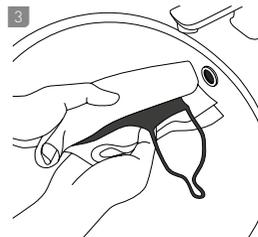


3 Pat the mask with a dry towel to remove moisture.

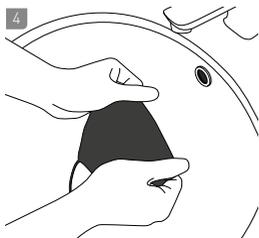

4 Straighten by grabbing ends of the mask by the nose and chin.

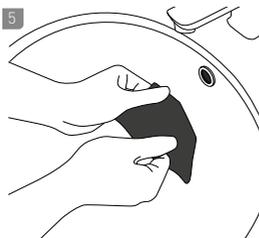

5 Straighten by spreading out the top and bottom sides of the mask.

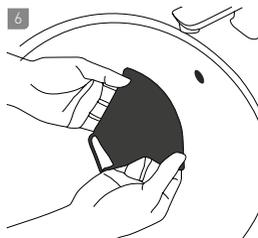

6 Lay the mask flat and dry in the shade.

- * The Inner filter is non-washable. Please remove the filter before washing the mask.
- * Remove all the air valve components, such as the air valve cover and the fan unit before washing.
- * Use a neutral detergent with cold water and wash gently. Lay the mask flat and dry in the shade.
- * Nose bridge may be damaged when an excessive force is applied.

